

Portex UniPerc Adjustable Flange Tracheostomy Tube

UR: _____

Surname: _____

Given Name(s): _____

Date of birth: _____

Affix Patient Label Here

TUBE INFORMATION


- The UniPerc Tracheostomy Tube is a longer tube with flange which can be adjusted by the Surgeon/Intensivist to suit the individual patient
- Used in patients who have a large amount of pre-tracheal tissue (large neck)
- The tube has a high volume, low pressure cuff which can be inflated/deflated with air
- Has a dedicated reusable (up to 29 days) inner cannula
- Inner cannula must be regularly removed and washed with the provided cotton swab




CLINICAL ALERTS

- Do NOT release blue locking clamp
- Releasing the locking clamp may cause the tube to fall out
- Measure the external tube length once per shift

Measurement _____ mm

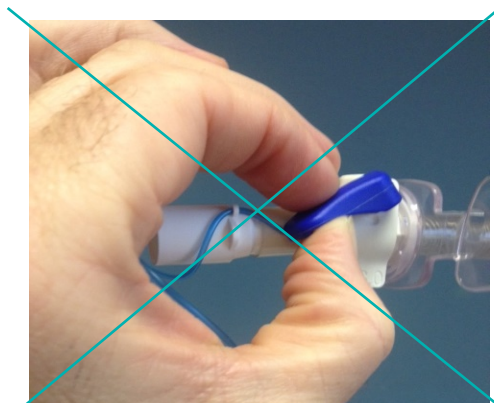

Measure and record external tube length from Tracheostomy hub in mm

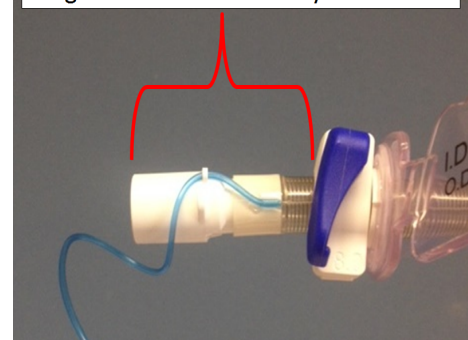

CONTACT

Name: _____ Pager: _____

or TRAMS pager 1291

Disclaimer: This Document has been developed for Austin Health use and has been specifically designed for Austin Health circumstances. Austin Health shall not be liable for any claims arising from the use of any information contained in this literature used by an organisation outside of Austin Health. Copyright © 2014 Austin Health. All rights reserved. Other clinicians and institutions are granted license to use this document and to modify it for their own purposes. It is a condition of this license that users acknowledge Austin Health TRAMS within all educational and promotional activities where this information is used.