

Use of the Bivona Aire-Cuf® Tracheostomy Tube

Patient's details:

UR: _____

Surname: _____

Given Name(s): _____

Date of birth: _____

Affix Patient Label Here

Clinical Instruction Sheet

Austin Health

Tube Information

- The Bivona Aire-Cuf® tracheostomy tube has an air-filled cuff. The pilot balloon is blue
- The silicone construction provides comfort for the patient
- The Bivona Aire-Cuf® is a single patient item that can be sterilized by CSSD and reused up to 10 times



Attention: Air-filled cuff

Cuff Inflation

- The cuff of the Bivona Aire-Cuf® tracheostomy is filled with air
- Measure the cuff pressure with a cuff pressure manometer (normal range 20-30mmHg)
- The Tracoe Smart Cuff Manager can also be used with this tube



Inner Cannula

- An inner cannula can be used with this tracheostomy tube and can be ordered from Sterile Stores
- Insert into tracheostomy tube. You should feel a slight snap once inserted in the correct position
- Change every 2-4 hours, or more frequently if necessary
- Clean with the purpose made sponge cleaner **not the brush type cleaner**. Discard the cleaning swab after 1 week of use
- Clean the inner cannula using tepid tap water with no detergent/soap. Allow to air dry
- The inner cannula should be discarded after 28 days or prior if kinked

Bivona inner cannula



Bivona cleaning swab

Contact

Clinician: _____ TEAMS or TRAMS phone x3095

Disclaimer: This Document has been developed for Austin Health use and has been specifically designed for Austin Health circumstances. Austin Health shall not be liable for any claims arising from the use of any information contained in this literature when used by an organisation outside of Austin Health. Copyright © 2022 Austin Health. All rights reserved. Other clinicians and institutions are granted license to use this document and to modify it for their own purposes. It is a condition of this license that users acknowledge Austin Health TRAMS within all educational and promotional activities where this information is used.